

Colostrum Collector

Protect Every Drop

Colostrum is a powerful therapy for the NICU infant and can lead to improved patient outcomes.¹ Every drop of colostrum is valuable and should be protected from loss. Medela's Colostrum Collector is a solution engineered to safely and securely, transport, store, and administer colostrum. The small container size sets realistic expectations, encouraging a pump-dependent mother to initiate her milk supply and provide colostrum to her baby.

Inspired by insights from those we serve

- **Improve patient outcomes:** By providing every drop of mother's own milk¹
- **Small size:** Encourages the initiation and building of milk supply for pump-dependent mothers
- **Reduce loss:** Colostrum is collected without entering the pump kit connector
- **Reliable:** Works seamlessly with Symphony® breast pump kits
- **Secure:** Snap-tight lid enables secure transportation and storage



How does the Colostrum Collector work?*

- 1 Colostrum Collector is attached to the Symphony® kit.
- 2 Mom uses the Symphony breast pump to initiate breast milk production.
- 3 Every drop of colostrum is collected in the collector.
- 4 Colostrum never reaches the rest of the kit, protecting it from loss and contamination.
- 5 After pumping, remove Colostrum Collector from Symphony kit and snap the lid onto the Colostrum Collector.



Product Specifications:

Product Name	Item Number	Description	Unit (Each Information)		Shipper Information		
			Dimensions (H x W x D Inches)	Weight (lbs)	Qty/Case	Dimensions (H x W x D Inches)	Weight (lbs)
Colostrum Collector 2-Pack	101044659	Includes 2 Colostrum Collectors and lids	2 x 5.5 x 4	0.06	40 packs of 2	16 x 10.875 x 9.25	2.4
Symphony Breast Pump Kit with Colostrum Collector	101044744	Includes 1 Symphony Initiation Kit and 4 Colostrum Collectors and lids	4 x 9.5 x 11.5	0.5	10 kits	22 x 12 x 19	5

Content appearance may vary.

¹Meier PP et al. J Pediatr. 2017;180:15-21.

*Refer to Instructions for Use for complete instructions.